
DYNACITY

1A510

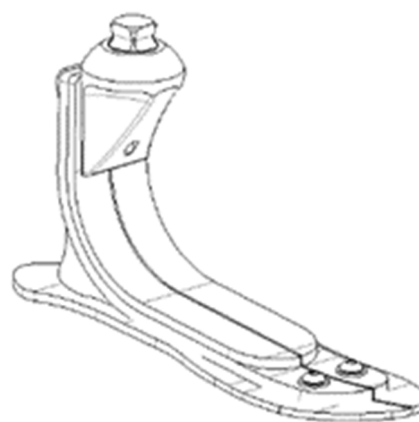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EN Instruction for use

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NO Brukerveiledning

SV Bruksanvisning

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SK Návod na používanie

HR Upute za uporabu

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DYNACITY - 1A510
Instructions for use for Prosthetists/Orthotists
Please read before using or working on the device

1A51099
2022-04

Pass on § 3, 7, 8 and 9 of these instructions to the Patient

1. COMPONENTS INCLUDED


Designation	Ref.	Included/Sold separately
Foot	1A510-xxx	Included
Foot shell	1A60010 pale skin colour or 1A60010-B brown skin colour	Included
Malleolar clip	1A40013, 1A40014 or 1A40015	Included
Black sock	1A40021-N	Included

2. DESCRIPTION, PROPERTIES AND MECHANISM OF ACTION

A. Description

An energy-returning prosthetic foot, designed to deliver safety and comfort, which patients can trust when performing everyday activities. On delivery, the following components are fitted:

- A prosthetic foot with a pyramid interface
 - A split carbon blade,
 - An intermediate split carbon blade,
 - A lower carbon plantar keel extending from the heel to the toes, split at the front,
 - A polyurethane foam wedge located behind the large blade on the plantar keel
- A polyurethane foot shell
- A sock woven from technical-grade fibres
- Malleolar clip

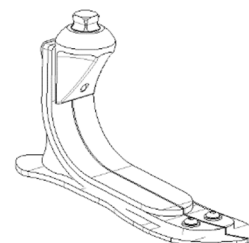B. Properties

Weight (Size 25)	445 g
Side	Right or Left
Size	22 to 30 cm
Maximum patient weight (Including carried load)	125 kg

This device has undergone a two-million-cycle test, in accordance with standard NF EN ISO 10328 (as indicated in the table below, up to a maximum patient weight of 125 kg), which corresponds to a service life of 2 to 3 years, depending on the patient's mobility level.

Category selection guide*					
Size	Patient weight (kg)				
	45 to 59	60 to 74	75 to 90	91 to 107	108 to 125
22	1	1	2	2	N/A
23	1	1	2	2	N/A
24	1	1	2	2	N/A
25	2	2	3	3	4
26	2	2	3	3	4
27	2	3	3	4	4
28	2	3	3	4	4
29	4	4	4	5	5
30	4	4	4	5	5
Module/Load level tested as per ISO 22675	-	-	-	-	4 / P6

*Categories are not the same as modules: weight alone is not sufficient to determine the category - foot size is also a factor.

C. Mechanism of action

At heel strike, only the rear part of the plantar keel is in contact with the ground. The deformation of the plantar keel and of the intermediate blade absorbs the loading that occurs at the start of the stance phase. The Poron wedge absorbs the contact noise or vibration generated at the interface between the large blade and the plantar keel.

As the stance phase progresses, the plantar keel flattens onto the ground; at midstance, only the intermediate blade is deformed, then the upper blade comes into play to allow a gradual roll-over until heel-off.

At heel-off, only the front part of the plantar keel is in contact with the ground; at toe-off, the energy stored when deforming these blades is returned to create an impulse.

The combination of the three split blades allows inversion/eversion of the foot.

3. INTENDED USERS/INDICATIONS

This medical device is supplied to healthcare professionals (prosthetists/orthotists) who then instruct the patient in how to use it. The prescription is drawn up by a doctor. The doctor works with a prosthetist/orthotist to assess whether the patient is suited to using the device.

⚠ This device should be used on **ONE PATIENT ONLY**. It must not be reused on other patients.

This device is exclusively intended to be used as orthopaedic equipment for an above-knee or below-knee amputee whose activity level is low-medium or normal (2 or 3).

Maximum weight (including carried load): 125 kg
Designed to accommodate a heel height of 0 to 20 mm.

4. CLINICAL BENEFITS

The device provides optimal comfort and safety for below-knee and above-knee amputees walking at normal speeds over smooth or uneven ground.

5. ACCESSORIES AND COMPATIBILITIES

The foot incorporates a male pyramid link that allows it to be used with female pyramid connectors (refer to our catalogue). The heel height can be adjusted from 0 to 20 mm.

6. ASSEMBLY AND FITTING TO THE PATIENT



A. Assembly

Place a block under the foot whose thickness corresponds to the height of the shoe's heel. Secure the female pyramid connector to the foot, positioning its axis perpendicular to the ground.

⚠ Tighten the connector's screws to the specified tightening torque.

B. Aligning

Align the device such that the load line passes through the posterior of the foot, 1/3rd of the length from the back.

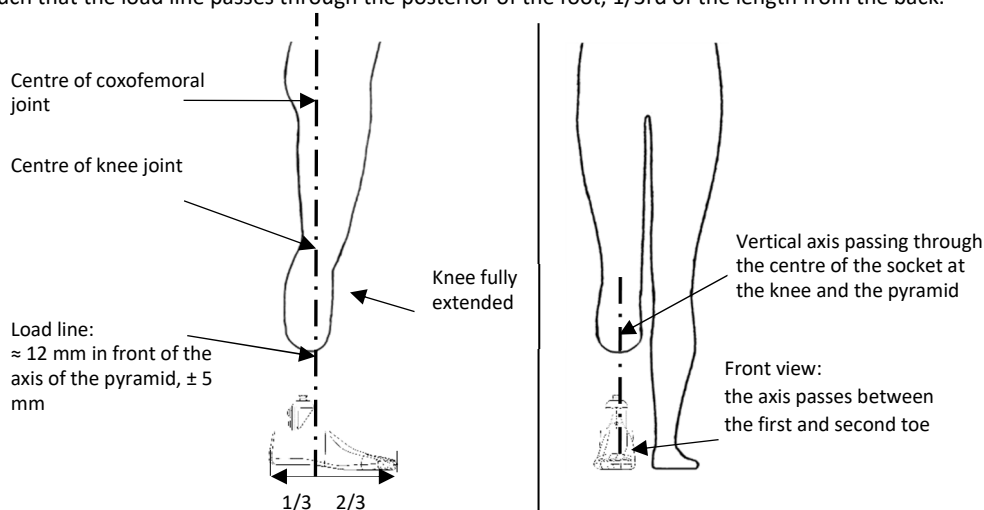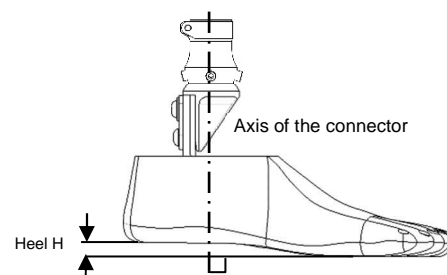- For a more dynamic alignment, increasing the plantar flexion improves the energy return at step completion.
- If, during roll-over, the patient feels that their foot is too long, or if the heel strike is too soft, the foot should be moved in the posterior direction with respect to the socket.

- If, during roll-over, the patient feels that their foot is too short, or if the heel strike is too rigid, the foot should be moved in the anterior direction with respect to the socket.

The dynamic alignment is performed in accordance with good professional practices.

C. Finishing

To avoid any rubbing noises, refit the sock up to the pyramid base and secure it in place using fine thread or something similar.

The malleolar clip provided allows the foot to be removed easily and quickly from the cosmetic foam of the calf prosthesis.

Comply with traditional good professional practices in terms of fitting the malleolar clip to cosmetic elements.

7. DETECTING ANY MALFUNCTIONING

- ⚠ If you notice that the device is behaving abnormally or if its characteristics feel different, or if it has been badly knocked, consult your prosthetist/orthotist.

8. WARNINGS, CONTRAINDICATIONS, SIDE EFFECTS

A. Warnings

- ⚠ Using the foot in a way that goes against the advice given by your prosthetist/orthotist may damage or degrade the foot's components (e.g. carrying heavy loads, excessive stresses, use after its service life has expired, etc.)

- ⚠ **The device is weatherproof only.**

B. Contraindications

- ⚠ This foot is not intended to be used for activities associated with a risk of significant impact or excessive loading.

- ⚠ This foot is not resistant to fresh water, salt water or chlorinated water.

C. Side effects

There are no side effects directly associated with the device.

Any serious incident that occurs which relates to the device must be reported to the manufacturer and to the competent authority of the member state in which the incident occurred.

9. MAINTENANCE, STORAGE, DISPOSAL AND SERVICE LIFE

A. Maintenance/cleaning

Clean the foot shell with a damp cloth or sponge.

- ⚠ All work on the foot should be performed by the prosthetist/orthotist; no other operations, such as lubrication, work on the fastenings or other work, need to be performed.

- ⚠ After accidental contact with water:

- Rinse the foot with clean water
- Dry thoroughly

- ⚠ The presence of sand, particles or foreign bodies inside the foot may modify its behaviour and damage it. After use in a highly soiling environment:

- Extract the foot from the foot shell using a shoehorn and remove the sock
- Rinse the foot and other components in clean water to remove any particles.
- Dry the various components before refitting the sock and reinserting the foot in the foot shell.
- Pull up firmly on the front of the sock to prevent it from getting caught between the moving parts of the foot.

- ⚠ The foot shell, sock and malleolar clip are the only components of the foot which may need to be replaced (in the event of wear, loss, etc.).

Foot shell:

- Use a shoehorn to remove and refit the foot shell.

- ⚠ Only use spare parts kit **1A60020-XYX** (X: Side/YY: Size (cm)) including foot shell, sock and malleolar clip.

B. Storage

Service and storage temperature: -20°C to +60°C

Relative humidity of the air: no restrictions

C. Disposal

The various components of the foot are considered as special waste: stainless steel, aluminium, carbon fibre, epoxy resin, thermoplastic (polyethylene), polyurethane. They must be treated in accordance with current, applicable legislation.

D. Service life

It is advisable to arrange for a prosthetist/orthotist to check the product once a year.

10. DESCRIPTION OF THE SYMBOLS

	Manufacturer		Identified risk		CE marking and year of first declaration of conformity
---	--------------	---	-----------------	---	--

11. REGULATORY INFORMATION

This product is a CE-marked medical device that is certified as conforming with regulation (EU) 2017/745

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PROTEOR SAS